

The assured and reliable sterilization brand from

Japan

HOSPITAL-SUPPLIES™

HP
SP

Protect Sterilization Pouches and Medical Instruments!



Product name	Product #	Dimensions(mm)	Content
Instrument fixing sheet	MP-01A	80*165	500 pcs
	MP-02A	50*135	
	MP-03A	32*127	

Distributor

INSTRUMENT FIXING SHEET

Easy to use and can protect important instruments

NiGK Corporation

Overseas Business Development Division

TOKYU BLDG. EAST NO.5

2-25-5 Minami-ikebukuro, Toshima, Tokyo, 171-0022 Japan

Tel. +81-3-3986-3910 (English)

Fax. +81-3-3983-8286

E-mail indicators@nichigi.co.jp

Website www.nichigi.co.jp/en




MADE IN JAPAN


©2020 NiGK Corp. All rights reserved

What are the applications of fixing sheet?


Prevent breaking sterilization pouch.

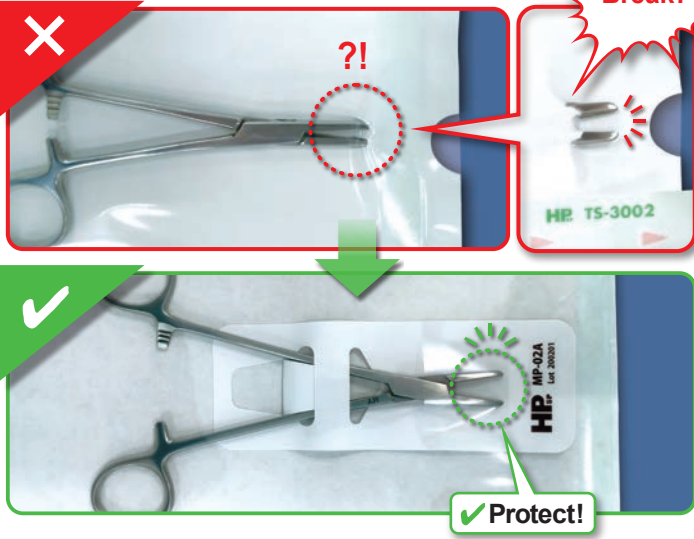

Keep the instruments in open position!

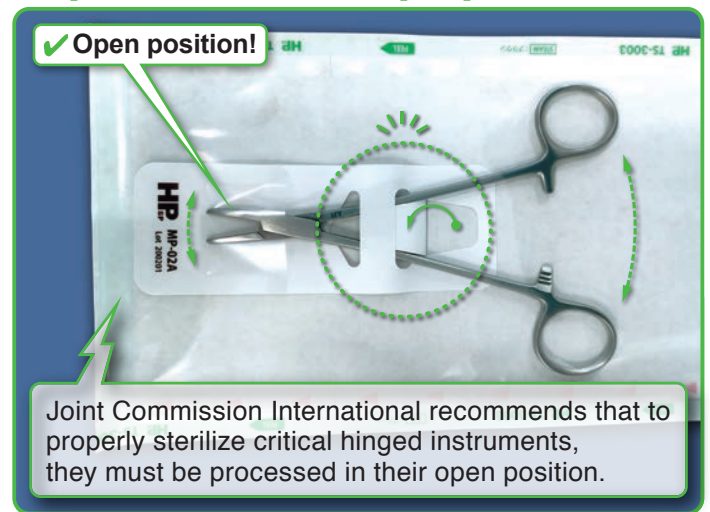

Easy to use, and easy to write the spec!

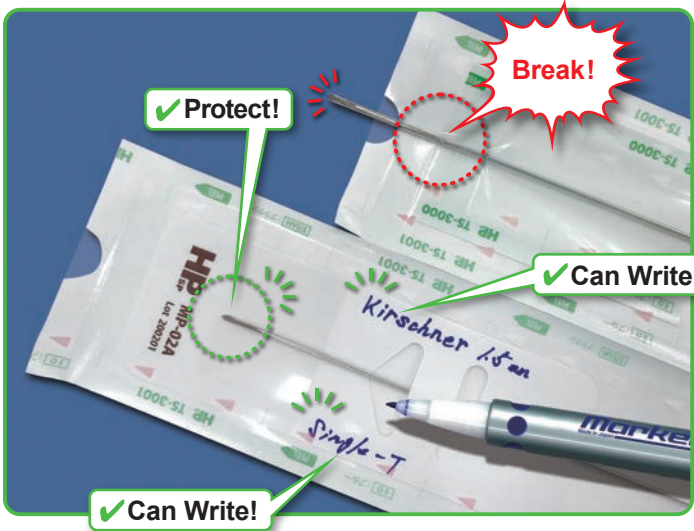


Protect the tip of important medical device.




Example of recommended medical instruments (for orthopedics, neurosurgery, ophthalmology and other subjects)




Kirschner wire

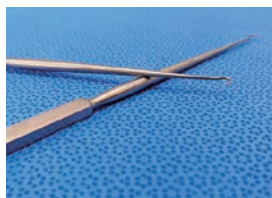

Skin hook

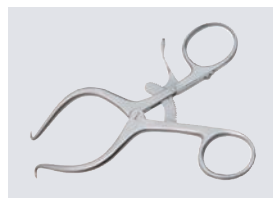

Gelpi retractor

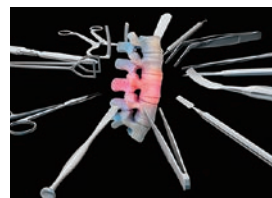

Chisel



laparoscopic surgery instruments

How to use

1. Please insert the tip of a medical device to the pocket.
(Refer to the instruction on the front of this package)
2. Run the sterilization process after packing this sheet into a sterilization pouch.

Precaution

1. Be careful with the tip of the medical device when using this product to avoid any injuries.
2. Please do not push the medical device strongly to the bottom of the pocket. Excessive load can lead to damage to devices or insufficient sterilization.
3. This product can be used for steam.
Do not use to other sterilizing processes and the other uses.

4. Do not peel a film in the pocket part from the body.
5. Avoid use at the place where many water (drain) occurs and collects.
6. This product is a disposable product. Do not reuse.

Storage

1. Please keep the products in a closed box in a clean room, away from direct sunlight, high humidity and high temperature.

Shelf life

1. The shelf life is 5 years from the production date.